

INTUBLADE

DISPOSABLE VIDEO LARYNGOSCOPE · LENS-CLEARING

QUICK START GUIDE

PLUGS INTO SUPPORTED USB-C DEVICES*

Compatible with supported iOS
and Android devices

LUER-LOCK PORT

Twist-lock saline syringe port

134 G TOTAL WEIGHT

Lighter than your phone

HIGH-TECH POLYMER CONSTRUCTION

Built to perform

PATENTED LENS-CLEARING TECH

Clears the view mid-intubation

COMPLETELY DISPOSABLE

Single use. No cleaning, no reprocessing

HD CAMERA

1280 × 720 HD
Video Feed

STANDARD MACINTOSH GEOMETRY

Familiar technique. No retraining required

1,500 LUX LED

High-intensity illumination for
improved visualization

**SEE CLEAR.
ACT FAST.
INTUBATE WITH CONFIDENCE.**

VETERAN-OWNED & OPERATED

Compatible with supported USB-C iOS and Android devices. See website for details.

INTUBLADE

QUICK START GUIDE

SETUP · TECHNIQUE · LENS CLEARING



ALWAYS CHECK THE
PATIENT'S AIRWAY.

DO NOT RELY SOLELY
ON THE SCREEN.

SETUP PROCEDURE

01

ATTACH & PREPARE

Choose the correct single-use blade (Size 3 or 4).
Confirm expiration date. Plug IntuBlade into your
iOS or Android device via USB-C.

02

APP OPENS & VERIFY

The IntuBlade app launches automatically on
connection. Confirm live video feed is visible and
clear before proceeding.

03

RECORDING (IN-APP)

Recording occurs within the app. Ensure the app is
open and recording is active before intubation. A brief
delay may occur after connection.

04

LOAD SALINE SYRINGE

Insert a 10 mL saline-filled syringe into the handle's
luer-lock port. Twist to lock. Press briefly to clear
the lens.

05

DISPOSE AFTER USE

Single use only. Do not reuse or sterilize. Dispose per
facility protocol.

INTUBATION TECHNIQUE

01

INSERT BLADE

Left hand. Insert midline. Sweep tongue left.
Standard Macintosh technique.

02

FIND THE EPIGLOTTIS

Advance to the vallecula. Lift anteriorly. Vocal
cords visualized on screen.

03

GUIDE THE TUBE

Maintain direct airway awareness. Do not rely
solely on the screen. Pass ETT under video
visualization.

04

CONFIRM + SECURE

Inflate cuff. Confirm with EtCO₂, auscultation, and
chest rise. Secure tube.

QUICK TIPS : SOILED AIRWAY

- PRESS SALINE PLUNGER BRIEFLY TO CLEAR LENS MID-INTUBATION.
- USE SALAD (SUCTION-ASSISTED LARYNGOSCOPY) FOR HEAVY SECRETIONS
- SUCTION ALONGSIDE BLADE AS NEEDED.
- LIMIT TO TWO ATTEMPTS. IF UNSUCCESSFUL: BVM → SUPRAGLOTTIC AIRWAY
→ ESCALATE PER PROTOCOL.



RESOURCES & TRAINING

[INTUBLADE.COM/RESOURCES](https://intublade.com/resources)
APP · SKILLS SHEET · GUIDE

SUPPORT

[SUPPORT@INTUBLADE.COM](mailto:support@intublade.com)

SALES

[SALES@INTUBLADE.COM](mailto:sales@intublade.com)

WEB

[INTUBLADE.COM](https://intublade.com)